200806-103

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HbA_{1c} Control for NORUDIA N HbA_{1c} kit

This product is the control for the assay to determine levels of hemoglobin A_{1c} in whole blood using **NORUDIA N HbA_{1c}** on an automated analyzer.

Preparation

1. Add 1mL of purified water to one vial of this product. Mix the contents of the vial for approximately 10 minutes by the wave-rotor or leave the vial to stand for 10 minutes at room temperature and then gently mix by swirling and turning upside down several times. The reconstituted control is stable for 10 days if refrigerated. Avoid freezing. Allow the refrigerated control to reach room temperature (15-30°C) before measurement.
2. The reconstituted control kept for more than 2 hours should be mixed well before use.
3. HbA_{1c} values are printed on the reverse.

Handling Precautions

1. This product is the control specified for measurement with **NORUDIA N HbA_{1c}**. Do not use this product for any other purpose.
2. Reconstitute this control with purified water at about 20°C as it may not be easily dissolved with cold water.
3. Human source material used in formulating this control has been confirmed to be negative for HBs antigens, HCV antibodies, HIV antibodies. However, handle this product in accordance with appropriate biohazard procedures as though capable of transmitting infectious agents. Wear disposable gloves to prevent disease transmission.

〔包装 (Packaging)〕 1mL用×2濃度×6 (Two concentrations x 6 vials for 1mL)

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Table for the concentration of HbA_{1c} Control for NORUDIA N HbA_{1c} kit

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The HbA_{1c} values are different from each production lot. Please confirm that the labeled Lot No. of each vial is consistent with the Lot No. in this sheet before testing.

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SEKISUI MEDICAL CO., LTD.