204446-100

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# CRP Calibrator(Nanopia) E

CRP Calibrator(Nanopia)E is used in the determination of C-reactive protein in serum or plasma using Nanopia CRP kit on an automated analyzer.

## Preparation

Ready-to-use

## Handling Precautions

1. This product is a calibrator for in vitro use with an automated analyzer. Do not use it for any other purpose.
2. Human material used in formulating this calibrator has been tested and found to be negative for hepatitis B virus surface antigens, anti-human immunodeficiency virus antibodies, and anti-hepatitis C virus antibodies. However, since transmission of infectious disease is still possible, handle with care. Please wear protective gloves when handling this solution.
3. CRP Calibrator(Nanopia)E contains sodium azide as a preservative. Because sodium azide can react with lead or copper pipes to generate highly explosive metal azides, dispose the solution with a large quantity of water. If the solution is ingested or comes into contact with the eyes or skin, rinse the affected area immediately with plenty of water and seek medical advice.

**Packaging**     6 × 2 mL

**Storage Temperature**     2–10°C

**Expiration Date**     Printed on the outside of the package.

**CRP Concentration**     Printed on the label of the vial.

**SEKISUI MEDICAL CO.,LTD.**

1-3, Nihonbashi 2-chome, Chuo-ku, Tokyo 103-0027 JAPAN